

# RECORDING MONTAGE INFORMATION

RECORDING MONTAGE NAME: MSLT\_2

**Recording Settings:**

CH #	Channel Label	Patient	Signal Type	Gain	Range	LFF	HFF	Line Filter
1	L EOG	PT 1	EOG	20	x1000	0.3	30	IN
2	R EOG	PT 1	EOG	20	x1000	0.3	30	IN
3	L CENT	PT 1	EEG	20	x1000	0.3	30	IN
4	R CENT	PT 1	EEG	20	x1000	0.3	30	IN
5	L OCC	PT 1	EEG	20	x1000	0.3	30	IN
6	R OCC	PT 1	EEG	20	x1000	0.3	30	IN
7	CHIN EMG	PT 1	Chin	20	x1000	10	30	IN
8	ECG	PT 1	EKG	200	x10	3	30	IN

**Type of Montage:**

1 Patient

**Display Views:**

Display Views for Patient 1  
STANDARD

1-A2

2-A1

5-A2

6-A1

9-A2

10-A1

13-14

3-4

**Selector Panel Settings**

1 - A2

2 - A1

5 - A2

6 - A1

9 - A2

10 - A1

13 - 14

3 - 4

JACKBOX INPUTS  
FOR MSLT

