

AIRFLOW SENSOR USAGE AND JACKBOX PLACEMENT

Use Dymedix disposable sensors (oral/nasal) for all volunteers, except those with mustaches or CPAP units. Plug sensor into connect cable and cable into jackbox in Channel 10 (Blue – G1, White – G2).

Volunteers with mustaches---Use Protech 2-Channel thermistor.

Nasal- Blue in G1, Black (grd) in G2 in Channel 9

Oral – White in G1, Black in G2 in Channel 10

Use jumper cable for black.

Volunteers with CPAP---Use SleepMate sensor with connect cable.

Cut end of sensor is Nasal. Plug into red end of connect cable and other end of cable into G1 in Channel 9. Black (grd) into G2 in Channel 9.

Plug Oral (uncut end) into blue end of connect cable and other end into G1 in Channel 10.
Black (grd) into G2 in Channel 10.