

RECORDING MONTAGE INFORMATION

RECORDING MONTAGE NAME: RESEARCH NPSG_2

Recording Settings:

CH #	Channel Label	Patient	Signal Type	Gain	Range	LFF	HFF	Line Filter
1	L EOG	PT 1	EOG	20	x1000	0.3	30	IN
2	R EOG	PT 1	EOG	20	x1000	0.3	30	IN
3	L CENT	PT 1	EEG	20	x1000	0.3	30	IN
4	L OCC	PT 1	EEG	20	x1000	0.3	30	IN
5	CHIN EMG	PT 1	Chin	20	x1000	10	30	IN
6	LEG EMG	PT 1	Leg	10	x1000	10	30	IN
7	SNORE	PT 1	Snore	5	x1000	3	30	IN
8	ECG	PT 1	EKG	200	x10	3	30	IN
9	NASAL FLOW	PT 1	Flow	10	x1000	0.1	30	IN
10	ORAL FLOW	PT 1	Flow	10	x1000	0.1	30	IN
11	NASAL PRES	PT 1	Flow	5	x1000	0.1	30	IN
17	RIB CAGE	PT 1	Effort					
18	ABDOMEN	PT 1	Effort					
19	SUM	PT 1	Effort					
33	POSITION	PT 1	Position					
34	SAO2	PT 1	SaO2					

Type of Montage:

1 Patient

Display Views:

Display Views for Patient 1
STANDARD
STAGING

Selector Panel Settings

- 1 - A2
- 2 - A1
- 5 - A2
- 9 - A2
- 13 - 15
- 7 - 8 or 11 - 12
- 16 - 17
- 3 - 4

JACKBOX INPUTS FOR NPSG

